


## U.PLUS<sup>®</sup>

**Disposable Needle Compatible With Latest Version of Bard™ Magnum™ Biopsy Gun.**



- Provided with special removable handle for easy insertion of the needle into the biopsy gun.
- The stylet's cavity is designed to collect the sample without any damage (lacerations or tearings) for a perfect specimen quality
- The cavity and the tip are both reinforced to avoid bendings.
- Centimetre markings facilitate precise depth placement.
- Plastic cannula hub with syringe connector for cytological examinations.
- The tip is treated to make it echogenic and facilitate its positioning under ultrasound guidance.
- Color coded for immediate diameter identification.
- Available in a wide range of sizes.

## Disposable Needle Compatible With Latest Version of Bard™ Magnum™ Biopsy Gun.

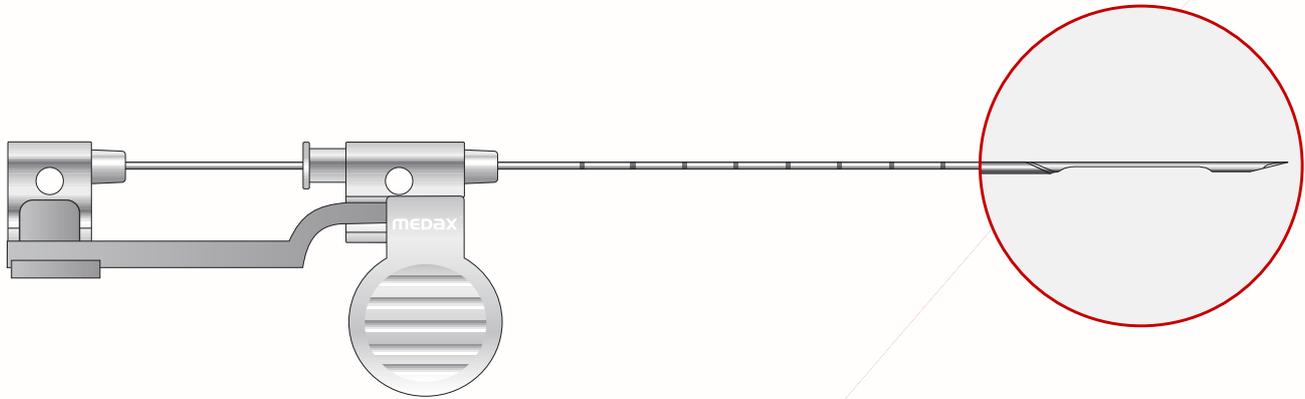

### Instruction for use:

1. After opening the blister, remove the plastic protection from the cannula.
2. Pick up the needle using the white little handle on the right side.
3. Insert the needle into the metal pivots of the gun.
4. Remove the right handle by sliding it off to your right (an arrow displays the correct direction).
5. Then remove the left handle together with the plastic support which connects the two plastic hubs of the cannula and of the stylet.
6. Close the cover of the gun.
7. Before use read carefully the instructions provided by the manufacturer of the gun.
8. After use pay attention to the necessary precautions for potentially contaminated materials.
9. To remove the needle from the gun, re-position the left handle first, so that the two hubs are connected by the plastic insert; then slide the right handle onto that, as its previous configuration. Using both your hands, grasp the handles and gently take the needle out of the gun.
10. Before discarding, cover the needle with its protective case.
11. Throw away after the use and discharge together with the hospital waste following, the laws in force.

		LENGTH IN MILLIMETRE							
DIAMETER IN GAUGE	Colour code	80	100	120	160	180	200	250	300
14	Dark green	UP 14 080-00	UP 14 100-00	UP 14 120-00	UP 14 160-00	UP 14 180-00	UP 14 200-00	UP 14 250-00	UP 14 300-00
16	White	UP 16 080-00	UP 16 100-00	UP 16 120-00	UP 16 160-00	UP 16 180-00	UP 16 200-00	UP 16 250-00	UP 16 300-00
18	Pink	UP 18 080-00	UP 18 100-00	UP 18 120-00	UP 18 160-00	UP 18 180-00	UP 18 200-00	UP 18 250-00	UP 18 300-00
20	Yellow	UP 20 080-00	UP 20 100-00	UP 20 120-00	UP 20 160-00	UP 20 180-00	UP 20 200-00	UP 20 250-00	UP 20 300-00
21	Light green	UP 21 080-00	UP 21 100-00	UP 21 120-00	UP 21 160-00	UP 21 180-00	UP 21 200-00	UP 21 250-00	UP 21 300-00

Distributed By

